



pharyn	▼
QuickStrepA	
pharyn	

Print

Select new sample. Enter patient information in the new window.

Select the test type **pharyn** or **QuickStrepA** (only group A streptococci).

Print bar code label and attach it on the mariPOC® sample tube.


1.



Use only flocked swabs for sampling.


Disinfect the scissors between samples.

2.

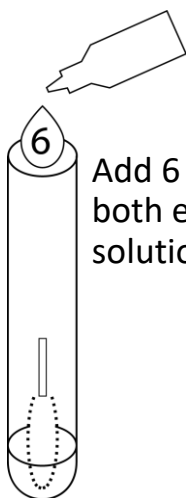

Add 6 drops of both extraction solution A and B.


3.

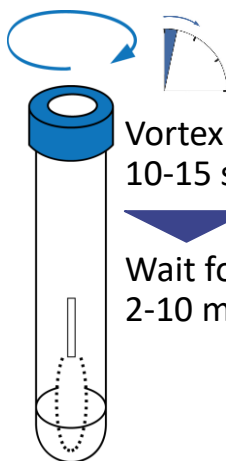

Vortex for 10-15 s.

Wait for 2-10 min.

4.

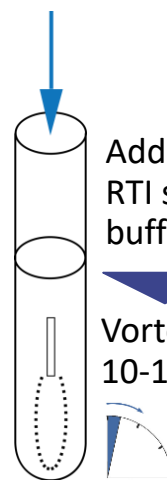


Add 1.3 ml of RTI sample buffer.

Vortex for 10-15 s.

Insert the sample tube in the analyzer. Patient information should be seen through the gap.

Close the sample hatch. Automated analysis starts.

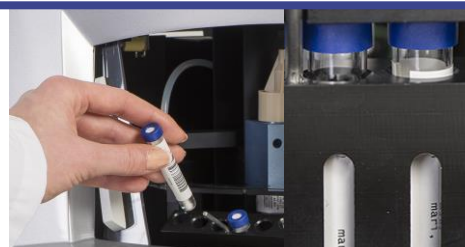

Results reporting

Pharyn

In 15 minutes: Strong positives and positives  
In 55 minutes: Low positives and negatives

QuickStrepA

In 15 minutes: Positives and negatives