


Select new sample. Enter patient information in the new window.

Select test type between **respi** or **respi < 7 years old** (without pneumococci).

Print bar code label and attach it on the mariPOC® sample tube.

1.

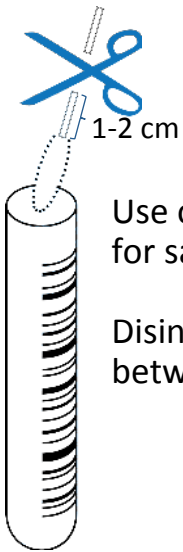


Use only flocked swabs for sampling.
Disinfect the scissors between samples.

2.



Add 1.3 ml of RTI sample buffer.

3.

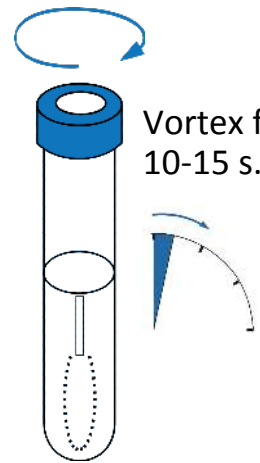

Vortex for 10-15 s.

Insert the sample tube in the analyzer. Patient information should be seen through the gap.

Close the sample hatch.
Automated analysis starts.



Results reporting

see user's manual for more information

- negative (reported only in 2 hours)
- +/- default negative (ψ is a probability for positivity)
- + positive (ψ is a signal multiple to cut off)