



Select new sample. Enter patient information in the new window.

Please select


Please select

- gastro + CDI
- gastroVir + CDI
- gastro
- gastroVir
- CDI

Select test type.



Print bar code label and attach it on the mariPOC[®] sample tube. Place the cap.

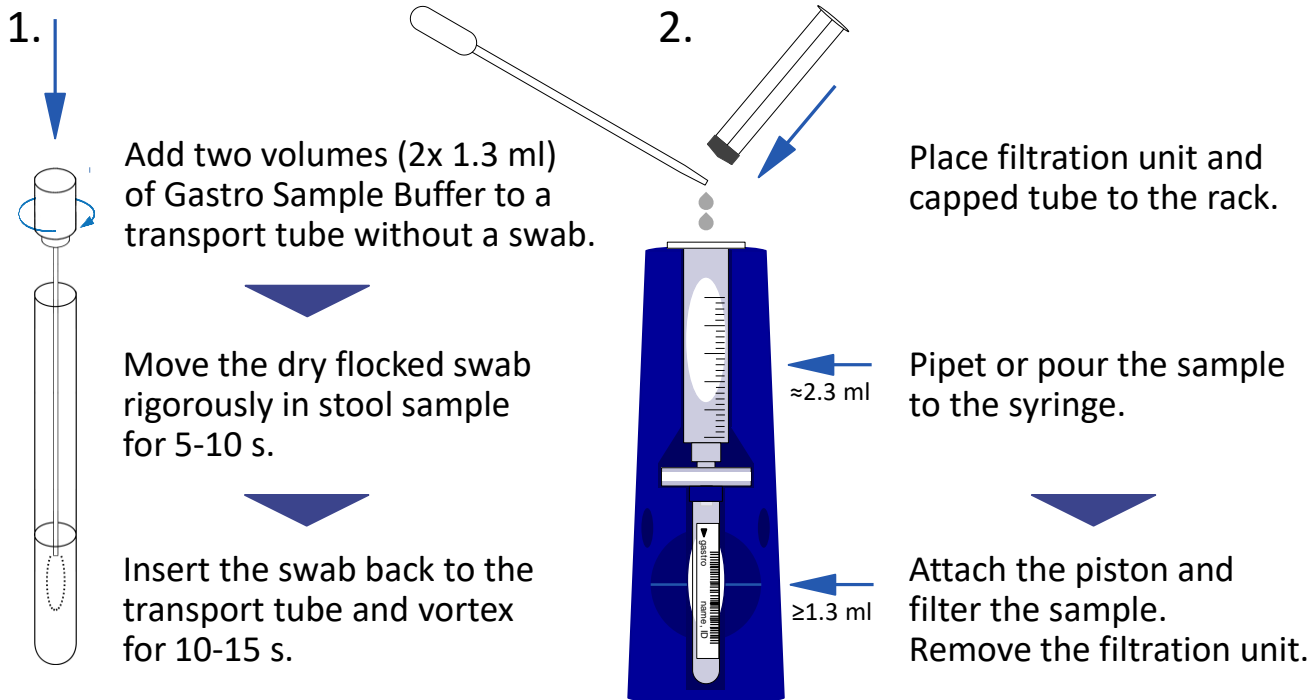

Insert the sample tube in the analyzer. Patient information should be seen through the gap.

Close the sample hatch. Automated analysis starts.



Results reporting

see user's manual for more information

- negative (reported only in 2 hours)
- + positive (ψ is a signal multiple to cut off)