



Select new sample. Enter patient information in the new window.

Please select

Please select


- gastro + CDI
- gastroVir + CDI
- gastro
- gastroVir
- CDI

Select test type.

Print

Print bar code label and attach it on the mariPOC[®] sample tube. Place the cap.

1.

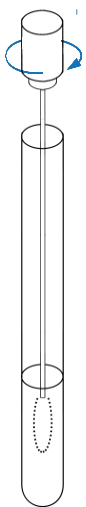


Add two volumes (2x 1.3 ml) of Gastro Sample Buffer to a transport tube without a swab.

Move the dry flocced swab rigorously in stool sample for 5-10 s.

Insert the swab back to the transport tube and vortex for 10-15 s.

2.

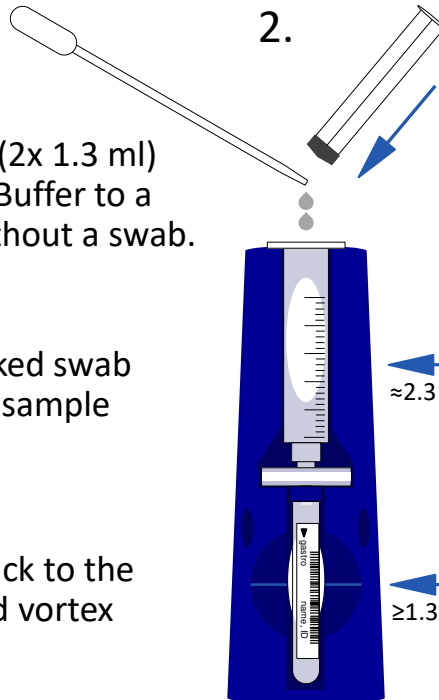

Place filtration unit and capped tube to the rack.

Pipet or pour the sample to the syringe.

Attach the piston and filter the sample. Remove the filtration unit.

Insert the sample tube in the analyzer. Patient information should be seen through the gap.

Close the sample hatch. Automated analysis starts.



Results reporting

see user's manual for more information

– negative (reported only in 2 hours)
+ positive (ψ is a signal multiple to cut off)