



Please select

Please select

- Gastro + CDI
- GastroVir + CDI
- Gastro
- GastroVir
- CDI


Print

Select new sample. Enter patient information in the new window.

Select test type.  
(The test type selection depends on the test plate in use.)

Print bar code label and attach it on the mariPOC® Sample tube. Place the cap.


1. Add two volumes (2x 1.3 ml) of Gastro sample buffer to a transport tube without a swab.

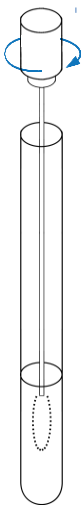

Move the dry flocced swab rigorously in stool sample for 5-10 s.

Insert the swab back to the transport tube and vortex for 10-15 s.

2. Place filtration unit and capped tube to the rack.

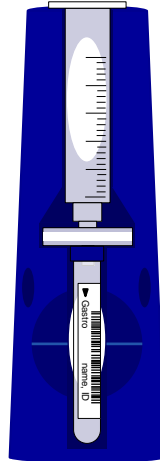

Place filtration unit and capped tube to the rack.

Pipet or pour the sample to the syringe.

Attach the piston and filter the sample. Remove the filtration unit.

Insert the sample tube in the analyser. Patient information should be seen through the gap.

Close the sample hatch. Automated analysis starts.



### Result Reporting

see user's manual for more information

Most of the positive results are reported in preliminary phase. **Negative results are reported** only in final phase.

mariPOC® test	Preliminary results	Final results
Gastro + CDI		
GastroVir + CDI		
Gastro, GastroVir	20 min: +	2 h: + or -
CDI		